



SAFETY AT WORK

Eyewash in case of foreign objects

Plum Eyewash is a sterile 0.9 % sodium chloride solution that eases discomfort and prevents following complications caused by foreign bodies in the eyes

PRODUCT FACTS & BENEFITS

- Effectively removes dust, dirt, and contamination in the eyes
- Easy to handle - easily applied by the injured person or with assistance
- Flexible solutions - bottles for mobile use and stations for wallmounting
- Ergonomic eyecups providing a soft and pleasant flow
- Wide application - from industrial workplaces to laboratories and first aid kits
- High functional reliability - bottles comply with to EN 15154-4
- Shelf life of 3.5 years for sealed bottles



The product is CE marked according to the Guidelines for Classification of Medical Device Directive (MDD 93/42/EEC).

info@plum.eu
+45 6916 9600

Combine Plum Eyewash with Plum pH Neutral for workplaces with a risk of chemical accidents

Know your chemicals and ensure the best suited first aid

The cause of an accident determines which eyewash is required. When an accident occurs, it is essential to determine whether it is caused by either a foreign object or by acids or alkali - and if possible, which pH value the substance in question holds.

When pH is either strongly acidic or alkalic, start rinsing with Plum pH Neutral until the bottle is empty. Discard the bottle and rinse with Plum Eyewash until you reach a doctor or the emergency room.

When pH is neutral, rinse with Plum Eyewash until the bottle is empty and discard the bottle after use.



4921 - Plum iBox 2 with 200 ml pH Neutral and 500 ml Plum Eyewash

ERGONOMIC EYECUPS

The bottles are equipped with a selection of ergonomic eyecups to ensure that the liquid can be applied correctly and in an even flow. Squeeze the bottle gently to generate a soft and pleasant flow.

MONO EYECUP



Flexible, ergonomic eyecup that keeps the eye lid open during use

DUO EYECUP



Flexible, ergonomic eyecup that allows rinsing of both eyes at the same time

AVAILABLE SIZES

Art. no.	4691	4604	4707	4861	4800
Size	200 ml	500 ml	1000 ml	500 ml	1000 ml
Packaging	10 x 200 ml	12 x 500 ml	6 x 1000 ml	6 x 500 ml	6 x 1000 ml

info@plum.eu
+45 6916 9600