Technical Data Sheet Plum Eyewash pH Neutral 4.9% Phosphate

	e product and the company		
1.1. Product identifier and i			
Trade Name:	Plum Eyewash pH Neutral 4.9% Phosphate		
Intended Use:	The products are intended to be used as first aid products for eye irrigation after accidents involv		
F	acids and alkali.		
Expiry date:	Printed on the bottle label.		
1.2. Details of the supplier o	f the data sheet		
Manufacturer			
Company:	Plum Safety ApS		
Address:	Mandelalleen 1		
Zip code:	5610		
City:	Assens		
Country	DENMARK		
E-mail	<u>info@plum.eu</u>		
Phone	+45 6916 9600		
Fax	+45 6916 9630		
1.3 Legislative information			
General information:	As the products are class 1s sterile medical devices according to Council Directive 93/42/EEC of		
	June 1993 concerning medical devices, they are certified by Notified Body, DNV Prod		
	Assurance AS.		
	No Safety Data Sheet is required for medical devices.		
	The products shall not be classified as hazardous according to the classification and labeling rul		
	for substance and mixtures.		
	Plum Safety ApS is certified according to ISO 13485:2016 Medical devices - Quality management syst		
	- Requirements for regulatory purposes.		
1.4 Instructions for use:	Mono eye cup (200 and 500ml) / Shower eye cup (1000 ml)		
	1 3 3 3 4 9 1 1 1 1 1 1 1 1 1 1 1 1 1		
	- Always wash with Plum Eyewash 0.9% Sodium Chloride after using Plum Eyewash pH Neutral 4.9% Phosphate!		
	- Do not reuse.		
	- Do not use if damaged.		
	- Recommended temperature when used		

Technical Data Sheet Plum Eyewash pH Neutral 4.9% Phosphate

2. General information about the product

2. General mormation about the product				
Material requirements:	The materials in the	hese products have been judg	ed biocompatible according the	
	•	0	of medical devices - Part 1: Evaluation and	
	testing within a risk manag	gement process.		
Bottle, label and eye cup dimensions.				
	000 11 11	500 11 11	1000 11 //1	

	200 ml bottle	500 ml bottle	1000 ml bottle
Bottle material	Polyethylene without additives	Polyethylene without additives	Polyethylene without additives
The bottle contains [ml]	201-207	505-512	1005-1020
Bottom Ø [mm]	58.5	78	90
Height [mm]	129.6	183	229

3. Side effects and malfunction

Side effects and malfunction:

The products are designed as a first aid product for eye irrigation. If side effects or malfunctions are experienced, please report these to Plum Safety ApS <u>info@plum.eu</u> or phone +45 6916 9600, so that necessary remedial measures can be initiated. Serious adverse effects should always be reported to a relevant and competent authority.

4. Storage		
Storage:	5°C	
Shelf life:	42 months from manufacturing date.	
Installation:	Please refer to the pictogramm on the label:	
Mounting stations and boxes:	Please visit https://plum.eu/products/firstaid/eye-wash	

5. Other information

Version history and indication of changes				
Version	Revision date	Responsible	Changes	
1.3	25-05-2021	CL	1.2, 1.3, 3, 5	
1.2	30-09-2020	CL	5	
1.1	03-06-2020	CL / VWB	3, 5	
1.0	14-04-2020	CL /VWB		

Other information Additional information can be obtained by contacting <u>info@plum.eu</u> or by telephone +45 6916 9600.

TDS prepared by:	
Company:	Plum Safety ApS
Address:	Mandelalleen 1
Zip code:	5610
City	Assens
Country:	DENMARK
E-mail:	info@plum.eu
Phone:	+45 6916 9600
Fax:	+45 6916 9630
Decument Lenguages	English
Document Language:	English