



SAFETY AT WORK

Plum Wound and Eyewash Plug & Heat Box

Dust-protected wall station in impact-proof polystyrene for quick and easy first aid when an eye accident occurs.

Whatever the reason, most often quick help is needed when you get something in wounds or eyes. Therefore, preparedness must be in place and be easily accessible at the workplace – also under cold weather conditions.

Plum Wound and Eyewash Plug & Heat Box contains a heating element, that ensures, that the eyewash solution has a comfortable rinse temperature.

PRODUCT FACTS & BENEFITS


- Dust-protected wall station in impact-proof polystyrene
- Power consumption: 20W
- Power supply: Connect to a 12V DC cigarette lighter or a local converter
- Heating foil is CE-approved in conformity with low voltage directive 2006/95/EC (EN 60950-1/A2:2013)
- The box should be placed sheltered from the influences of the weather (rain, snow, sun, etc.)
- The box is preset to ensure an eyewash temperature of 16°C to 38°C within an external temperature range of -20°C to +15°C
- If a transformer is used, it must meet the applicable local requirements for insulation and overcurrent protection
- Refills are not included

ELECTRIC CONNECTION

- If a transformer is used, it must meet the applicable local requirements for insulation and overcurrent protection
- Plum Safety ApS cannot be held liable for improper choice of transformer resulting in electric shock or other hazards created hereby

PLEASE BE AWARE!

- If the thermostat is detached from the heating foil, the whole box must be replaced. Otherwise, the content of the bottles is at risk of overheating
- We recommend checking the box temperature minimum 4 times a year. This can be done by simply feeling if the bottles are lukewarm (~ 20°C)

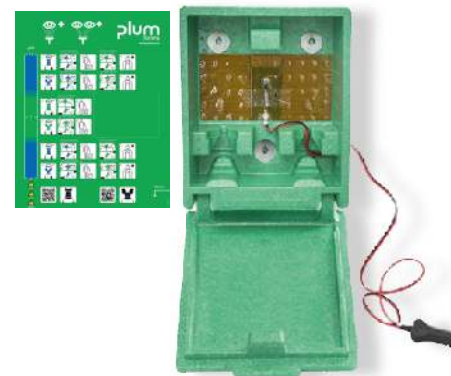

Plum Wound and Eyewash Plug & Heat Box

Suitable for 2 bottles of 200 ml and/or 500 ml wound and eyewash.

Art. No.	4672
Dimension	h:270 x w:225 x d:110 mm
Delivered empty. Transformer not included.	

info@plum.eu
+45 6916 9600