# Plum BodyNeutrAll

## 100ml, 600ml & 5L

Mobile body shower for first aid in case of chemical accidents  
Mobile Körperdusche zur Ersten Hilfe bei Chemikalien-Unfällen  
Douche mobile pour les premiers secours en cas d'accident chimique

Product information - in English - [page 1-2](#)

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# Plum BodyNeutrAll

Mobile body shower with a highly effective decontamination solution for skin, surfaces, equipment, and clothing.

## What is Plum BodyNeutrAll?

Plum BodyNeutrAll is an emergency spray for use on uninjured skin, clothing, and objects by instructed persons.

It is a medical device certified in the European Union for decontamination on uninjured skin and clothing after contact with oxidizing agents, acids, bases, aldehydes, hydrofluoric acid, and fluorides.

## How does Plum BodyNeutrAll work?

Plum BodyNeutrAll contains ascorbic acid, an amino acid buffer, and calcium ions as active substances. Glycerin, polyethylene and glycol are solubilizers for organic chemicals.

Plum BodyNeutrAll:

- reduces strong oxidants (such as chlorine, hypochlorite, chlorine dioxide, chloramine, bromine, iodine, fluorine, hydrogen peroxide, peracetic acid) to compounds that are no longer active.
- binds aldehydes into imines, phenols into amines, and heavy metals into ammine complexes.
- binds hydrofluoric acid and fluorides into the sparingly soluble calcium fluoride.
- dissolves organic solvents and chemicals.

A red coloration of the sprayed area shows that oxidizing agents have been inactivated.

**Instruction for use:** Spray affected areas of skin, clothing, equipment, and surfaces completely within the first two minutes. Allow 30 seconds to act. If possible, rinse with water afterwards.

## Are there any restrictions on use?

Plum BodyNeutrAll can be used without hesitation even if the chemical in question is not known. Plum BodyNeutrAll actively inactivates or neutralizes acids, alkalis, oxidizers, fluorides, aldehydes, and phenols. Organic solvents and chemicals as well as heavy metals are dissolved and removed from the uppermost layer of the skin and textiles by solubilizers and the hypotonic preparation. There are no restrictions for the application.

**Avoid eye contact!** Rinse with Plum Eyewash solution and seek medical advice. If possible, show the bottle label.

Do not use the product in eyes, on mucous membranes, or on injured skin. Do not use for accidents involving concentrated sulfuric acid. In the event of such contact, flush with water or sterile flushing solution

## Are there any side effects?

Plum BodyNeutrAll has been dermatologically tested for its skin compatibility. It shows no undesirable skin reactions (allergies) and no skin irritating effect if the instructions for use are observed. If undesirable effect is observed, the safety representative should be contacted immediately:

Plum Safety ApS +45 69 16 96 00 or [info@plum.eu](mailto:info@plum.eu).

# Plum BodyNeutrAll

## Is Plum BodyNeutrAll a hazardous substance or dangerous goods?

Plum BodyNeutrAll is not flammable, and not corrosive or harmful to health and therefore not a hazardous substance or dangerous goods.

## How to dispose Plum BodyNeutrAll?

Unused solution and incomplete emptied bottles to be disposed according to official regulations. Waste code: 200130 (Municipal waste, cleaning agents). Packaging to be handled according to 97/129/EC.

**Do not dispose the content of the bottles in wastewater!**

FEATURES	Plum BodyNeutrAll 5L – Art. No. 4735	Plum BodyNeutrAll 600ml – Art. No. 4736	Plum BodyNeutrAll 100ml – Art. No. 4737
Dimensions	Height: 570 mm, 800 mm with lance Width 220 mm Ø180 mm	Height: 290 mm Width: 110 mm Dept: 60 mm	Height: 130 mm Width: 65 mm Dept: 31 mm
Weight	10.0 kg	1.0 kg	0.175 kg
Yield	About 50 m <sup>2</sup>	About 6 m <sup>2</sup> or 3 persons	About 1 m <sup>2</sup> or 1 arm's length
Storage temperature	From 3° C to 35° C	From 3° C to 35° C	From 3° C to 35° C
Shelf life	3 years from date of production	3 years from date of production	3 years from date of production
Note	A red coloration of the sprayed area shows that oxidizing agents have been inactivated	A red coloration of the sprayed area shows that oxidizing agents have been inactivated	A red coloration of the sprayed area shows that oxidizing agents have been inactivated

[Go to our web](#) and read more about BodyNeutrAll

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