

Technical Data Sheet

Pull1Aid® BurnGel

1. Identification of the product and the company

1.1. Product identifier and intended use

Trade Name:	Pull1Aid® Burn Gel
Intended Use:	Plum Pull1Aid® Burn Gel is a sterile dressing with cooling gel intended to be used for burns.
Expiry date:	Printed on the sealed package.

1.2. Details of the supplier of the data sheet

Distributor	
Company:	Plum Safety ApS
Address:	Mandelalleen 1
Zip code:	5610
City:	Assens
Country	Denmark
E-mail	info@plum.eu
Phone	+45 6916 9600
Fax	+45 6916 9630

1.3 Legislative information

As the products are class IIb sterile medical devices according to *Council Directive 93/42/EEC of 14 June 1993* concerning medical devices, they are certified by the Notified Body, Intertek Semko AB, Kista, Sweden.
No Safety Data Sheet is required for medical devices.



Plum Safety ApS is certified according to *ISO 13485:2016 Medical devices – Quality management systems – Requirements for regulatory purposes*.

AKLA AB, Parkgatan 15, 696 33 Askersund, Sweden is the legal manufacturer of Pull1Aid® BurnGel.

2. General information about the product

Material requirements:	The materials in these products have been judged biocompatible according to the requirements of <i>ISO 10993-1:2018 Biological evaluation of medical devices – Part 1: Evaluation and testing within a risk management process</i> .
------------------------	--

Physical properties and composition:

	Pull1Aid® BurnGel Art. No. 5156 (REF. 94373)
Material	<i>Polyurethane, foam compress with 50g gel.</i> <i>Ingredients: Purified water, Melaleuca Oil (1%), Emulsifiers and preservatives</i>
Primary Packing (Multi layered pouch)	
Secondary Packing (Box)	
Dimensions	<i>10 x 10 cm</i>
Special feature	<i>Can be used as first aid on all degrees of thermal burns.</i> <i>Suitable as first aid on all skin areas. In case of eye contact rinse with water!</i>

Technical Data Sheet

Pull1Aid® BurnGel

3. Side effects and malfunction

Side effects and malfunction:

The products are designed for skin contact and first aid use involving burns.

If side effects or malfunctions are experienced, please report these to Plum Safety ApS, info@plum.eu or phone +45 6916 9600, so that necessary remedial measures can be initiated.

Reporting of accidents and incidents. Vigilance. Serious incidents involving products made available on the EU market must be reported immediately to the National Competent Authority. All serious incidents must be reported no later than 15 days after discovery, in the event of a serious threat to public health within 2 days. In the event of death or an unexpected and serious deterioration of a person's state of health, reporting must take place within 10 days.

4. Storage

Shelf life: 5 years from production date

5. Other information

Do not reuse



Do not use if package is damaged



Version history and indication of changes

Version	Revision date	Responsible	Changes
1.1	08-07-2025	MAW	1.3 + 2
1.0	20-06-2022	CL	

Other information Additional information can be obtained by contacting info@plum.eu or by telephone +45 6916 9600.

TDS prepared by:

Company: Plum Safety ApS
Address: Mandelalleen 1
Zip code: 5610
City: Assens
Country: DENMARK
E-mail: info@plum.eu
Phone: +45 6916 9600
Fax: +45 6916 9630

Document Language: English