

# Technical Data Sheet

## Plum QuickSoft Cohesive Bandage

### (blue, beige, black)

#### 1. Identification of the product and the company

##### 1.1 Product identifier and intended use

|               |                                               |
|---------------|-----------------------------------------------|
| Trade Name:   | Plum QuickSoft Cohesive Bandage               |
| Intended Use: | Intended to protect fingers and other joints. |
| Expiry date:  | Printed on the paper roll.                    |

##### 1.2 Details of the supplier of the data sheet

|              |                                                |
|--------------|------------------------------------------------|
| Manufacturer |                                                |
| Company:     | Plum Safety ApS                                |
| Address:     | Mandelalleen 1                                 |
| Zip code:    | 5610                                           |
| City:        | Assens                                         |
| Country:     | Denmark                                        |
| E-mail:      | <a href="mailto:info@plum.eu">info@plum.eu</a> |
| Phone:       | +45 6916 9600                                  |
| Fax:         | +45 6916 9630                                  |

##### 1.3 Legislative information

|                      |                                                                                                                                                                                                                                                                                           |
|----------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| General information: | As the products are class I medical devices according to <i>Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices</i> concerning medical devices. No notified body is involved. No Safety Data Sheet is required for medical devices. |
|----------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

#### 2. General information about the product

##### Physical properties and composition:

|                                  | Blue                                                       | Beige                                                      | Black                                                      |
|----------------------------------|------------------------------------------------------------|------------------------------------------------------------|------------------------------------------------------------|
| Material                         | 60 % Polyester<br>Polyurethane Foam<br>40% synthetic latex | 60 % Polyester<br>Polyurethane Foam<br>40% synthetic latex | 60 % Polyester Polyurethane<br>Foam<br>40% synthetic latex |
| Healthy Effects                  | May cause skin irritation                                  | May cause skin irritation                                  | May cause skin irritation.                                 |
| Physical and Chemical Properties | Flash point 470 <sup>0</sup> C                             | Flash point 470 <sup>0</sup> C                             | Flash point 470 <sup>0</sup> C                             |
| Primary Packing (Pouch)          | Packed with 2 layers of paper                              | Packed with 2 layers of paper                              | Packed with 2 layers of paper                              |
| Dimensions                       | 5.5 cm x 5 m                                               | 5.5 cm x 5 m                                               | 5.5 cm x 5 m                                               |
| Special feature                  | Latex free<br>Easy tear<br>Cohesive/self-adhesive          | Latex free<br>Easy tear<br>Cohesive/self-adhesive          | Latex free<br>Easy tear<br>Cohesive/self-adhesive          |

# Technical Data Sheet

## Plum QuickSoft Cohesive Bandage

(blue, beige, black)

### 3. Side effects and malfunction

#### Side effects and malfunction:

Since the product is not sterile it should not be used on large wounds and skin disease without a suitable sterile dressing. This product has a certain elasticity; therefore, it requires appropriate pressure and several layers when binding or fixing. Not too tight or too loose. Too tight will cause discomfort or adverse reactions to the user, such as local ischaemia. Too loose, will not hold the dressing in place or wrap it effectively.

The product is designed to protect fingers and other joints. If side effects or malfunctions are experienced, please report these to Plum Safety ApS, [info@plum.eu](mailto:info@plum.eu) or phone +45 6916 9600, so that necessary remedial measures can be initiated.


Serious adverse effects should always be reported to a relevant and competent authority.

### 4. Storage

#### Storage:



Keep dry

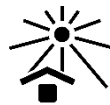

Keep away from sunlight



Storage temperature

#### Shelf life:

3 years from production date

### 5. Other information

#### Version history and indication of changes

| Version | Revision date | Responsible | Changes      |
|---------|---------------|-------------|--------------|
| 1.2     | 13-07-2026    | MAW         | 3, 5         |
| 1.1     | 18-12-2024    | MAW         | 1.1 + header |
| 1.0     | 14-02-2024    | LBH         |              |

#### Other information

Additional information can be obtained by contacting [info@plum.eu](mailto:info@plum.eu) or by telephone +45 6916 9600.

#### TDS prepared by:

**Company:** Plum Safety ApS  
**Address:** Mandelalleen 1  
**Zip code:** 5610  
**City:** Assens  
**Country:** Denmark  
**E-mail:** [info@plum.eu](mailto:info@plum.eu)  
**Phone:** +45 6916 9600  
**Fax:** +45 6916 9630

**Document Language:** English